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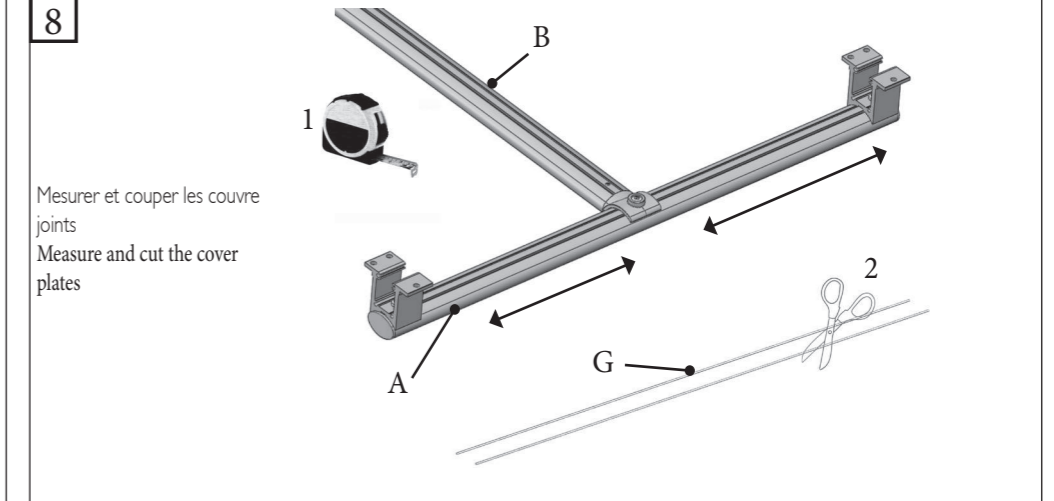
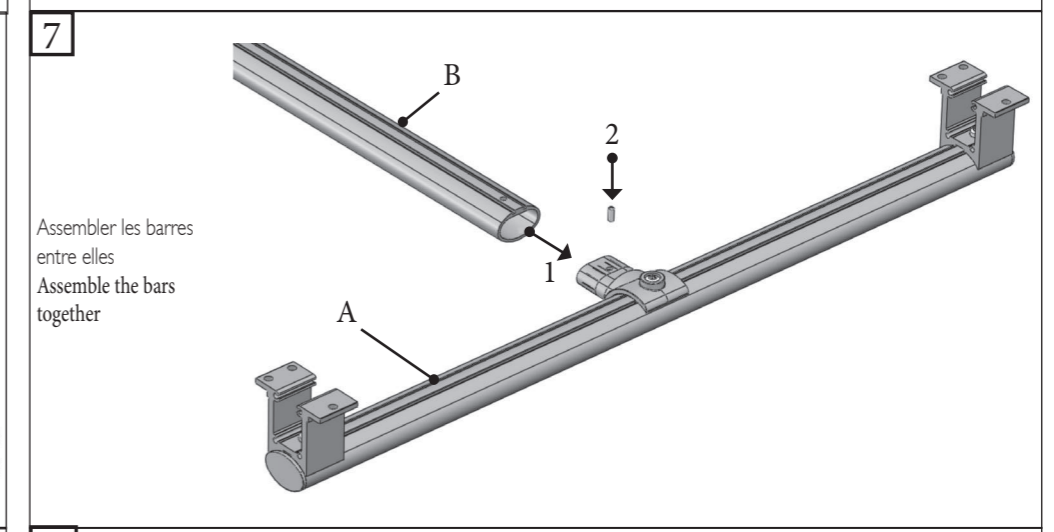
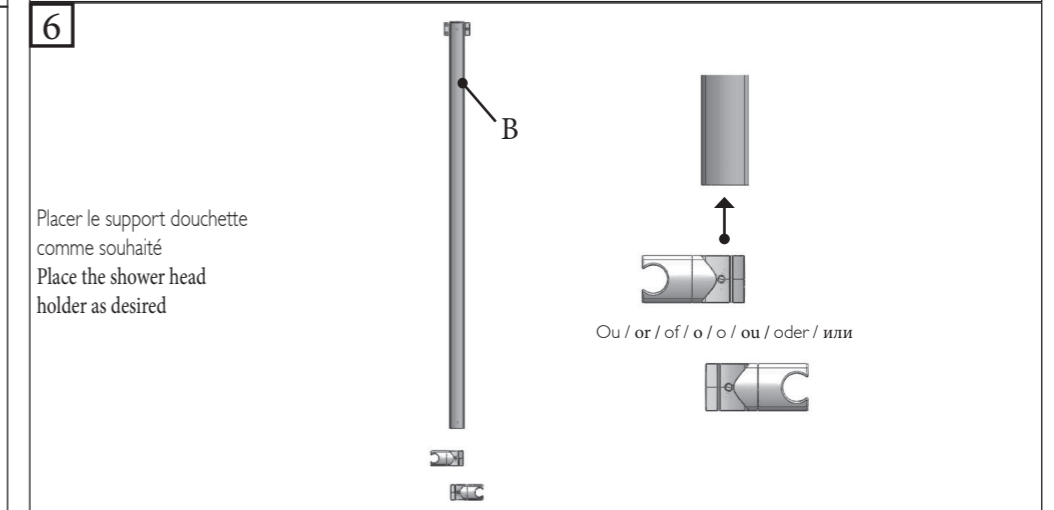
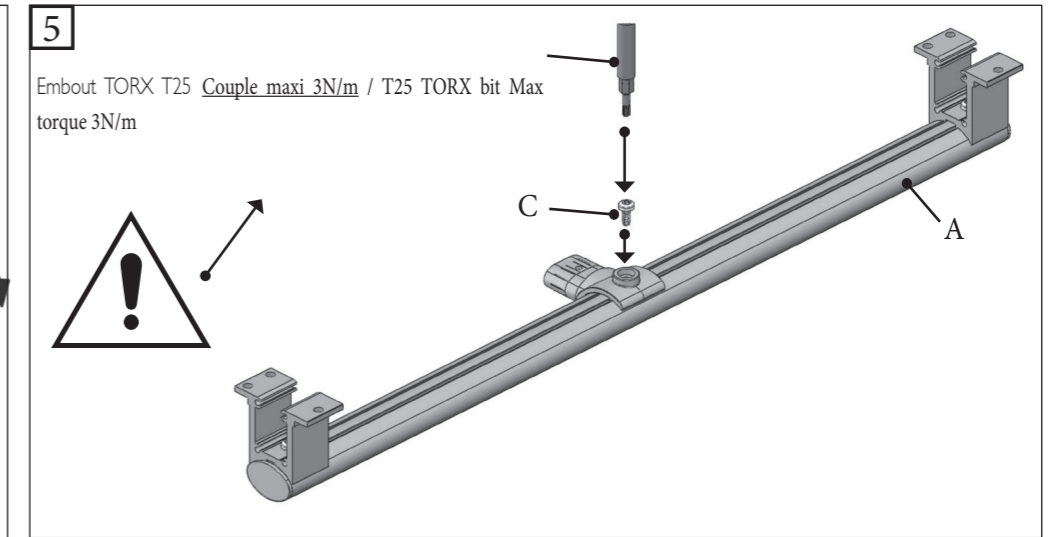
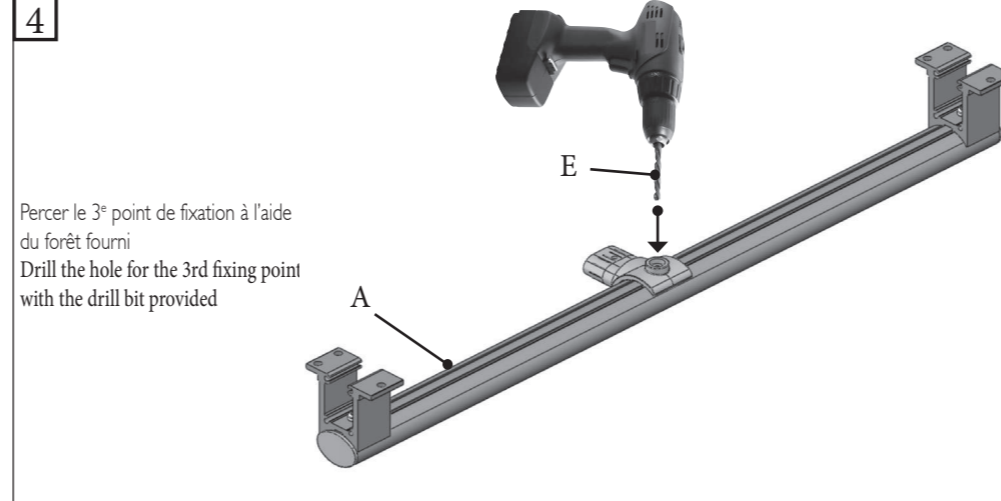
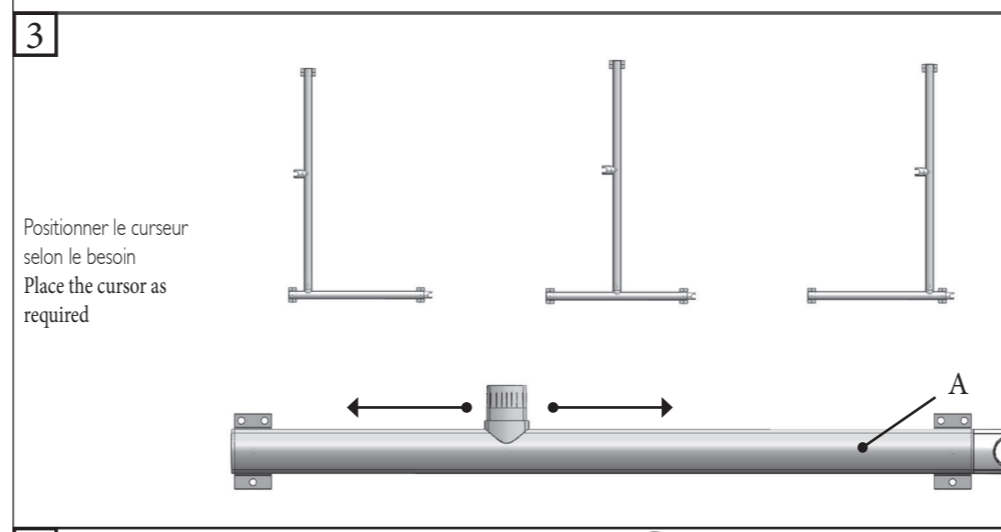
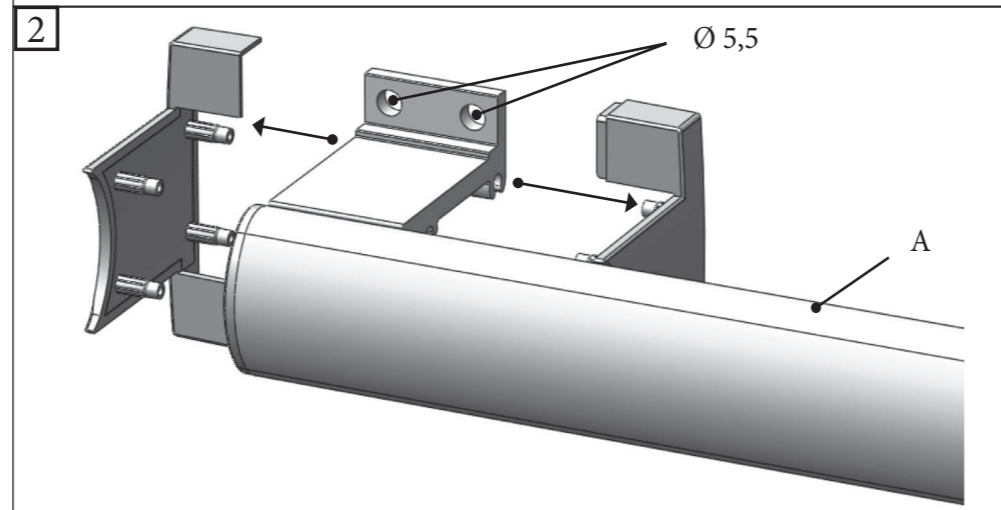
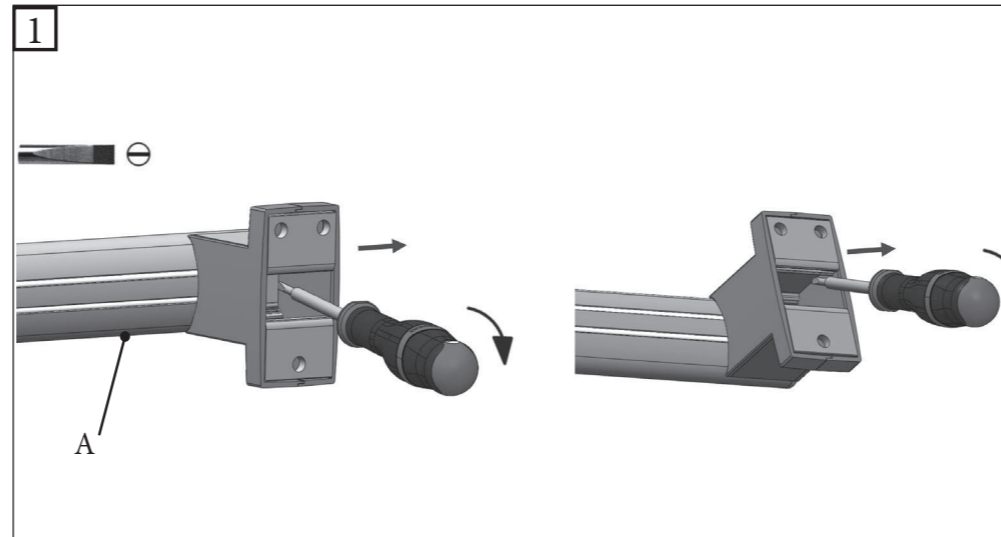
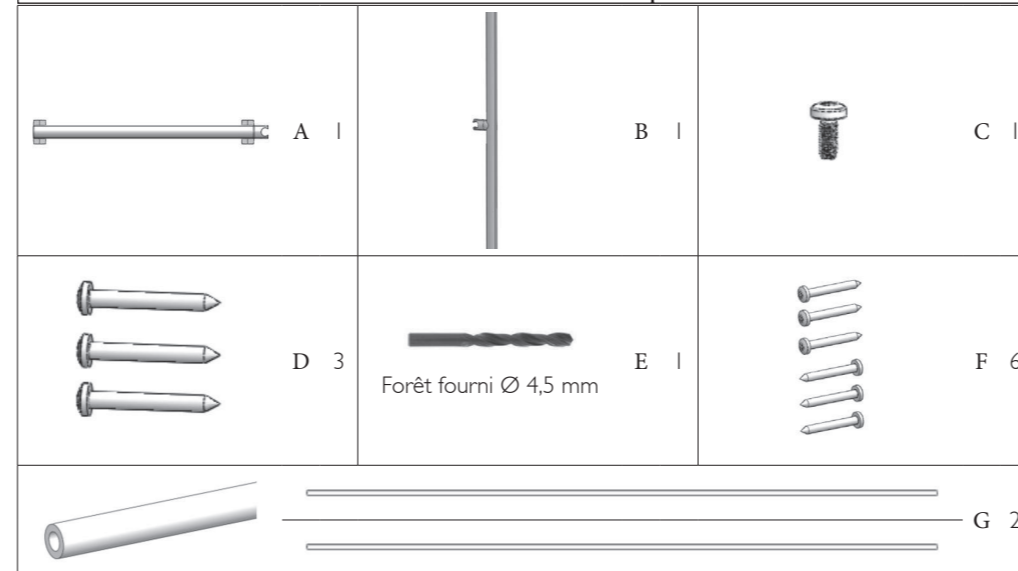
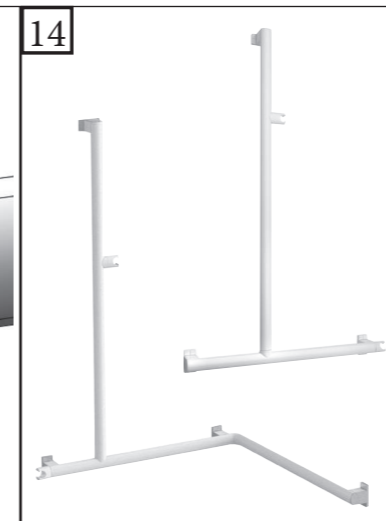
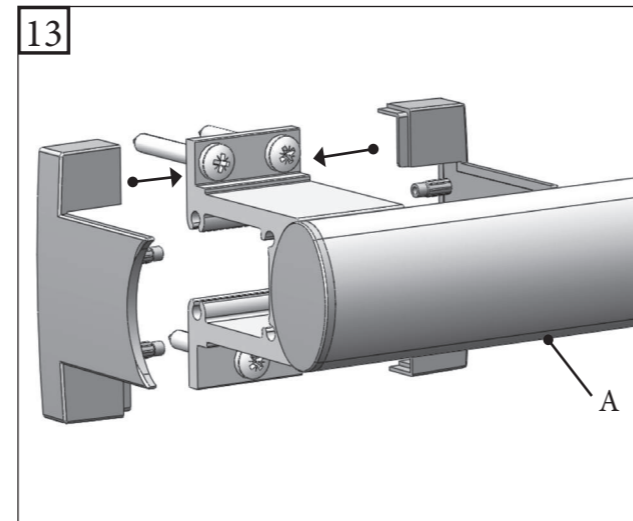
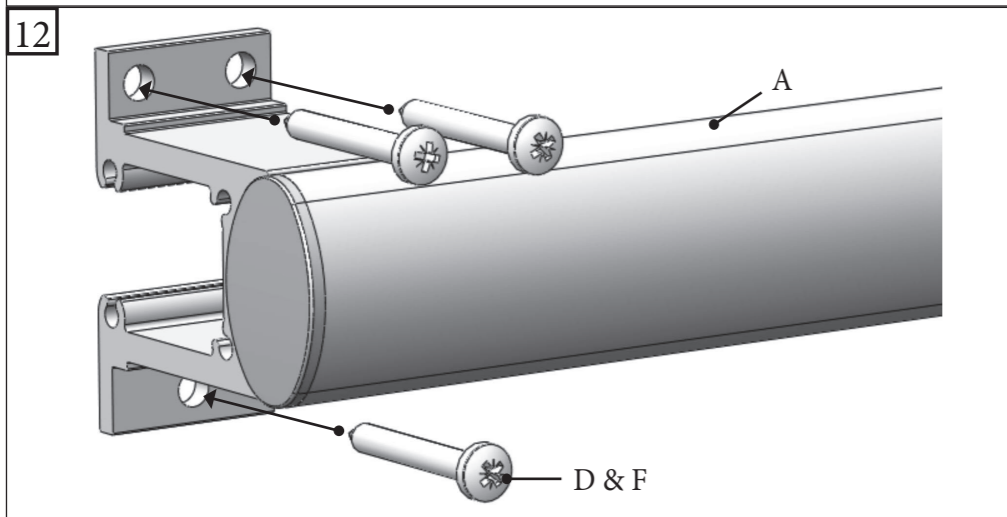
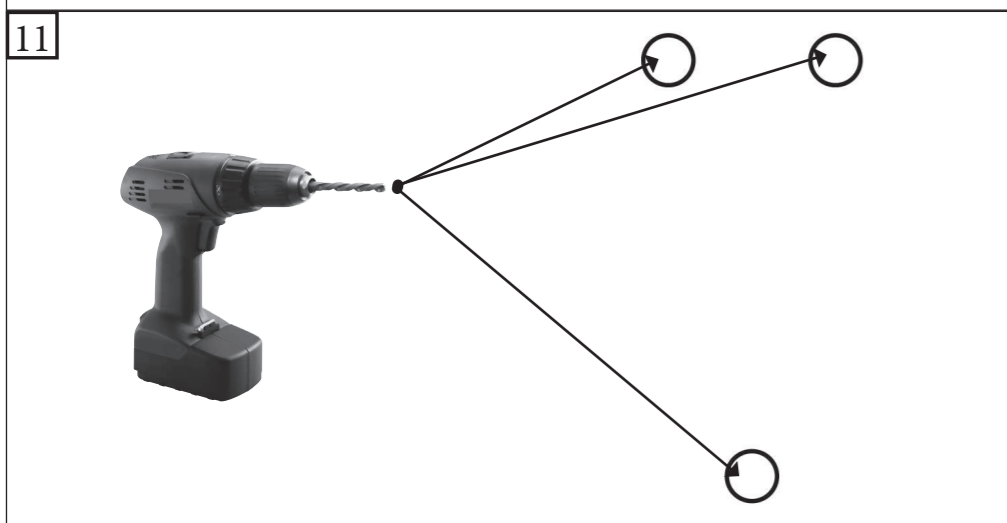
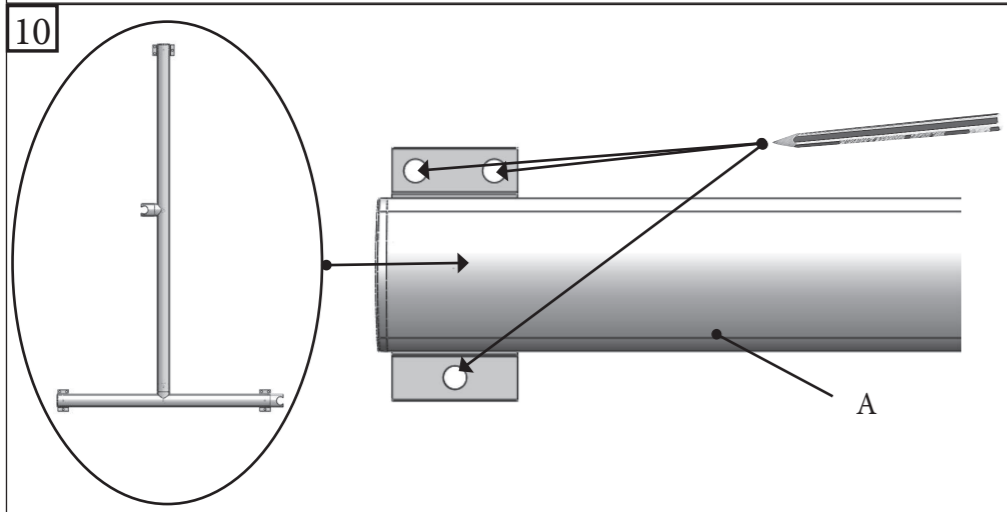
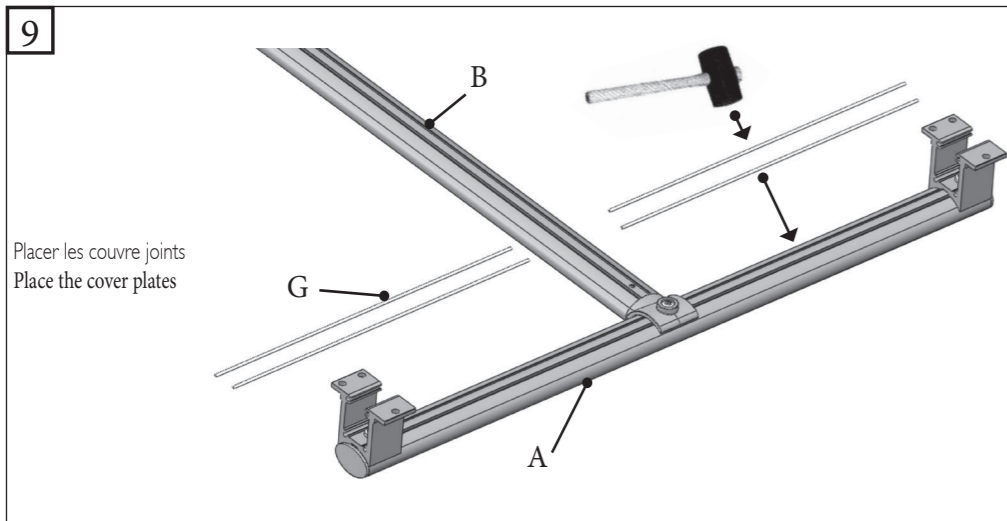
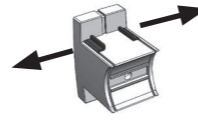
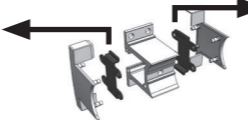
Do not install this equipment before reading the following instructions:

Fitting instructions. Advised height 80 cm from the ground. Before drilling into the wall, physically hold the product against the wall and mark the position of the holes to be drilled. Plugs have to be selected according to the type of wall. The fixings points have to be necessarily respected. **Care and maintenance.** IMPORTANT: IMPORTANT: In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of ketones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, rinse with clean water. **⚠ Safety warnings.** Grab bars add safety to the user if they are fitted according to instructions. After fitting and before use, make sure that the fixing screws are tightened securely. **DO NOT** use grab bar as a step to reach area, do not sit or hang on grab bar; use it for assistance ONLY. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this grab bar. **♻** At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

DEMONTAGE DES ENTRETOISES / DISMOUNTING SPACER	
1	2
	
Démontez l'entretoise de son support Remove the dismounting spacer from its bracket	Jeter l'entretoise, se reporter aux consignes de recyclage local Discard spacer, refer to local recycling locker

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